




## Sippi™ Quick Guide

# Starting Sippi™ after connecting a new patient

For continuous management of Sippi – see reverse.


### 1. Hang up Sippi on the bed.

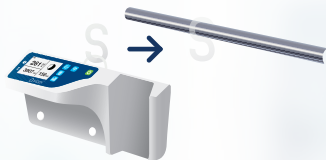

### 2. Start Sippi.



### 3. Attach a new drainage bag with cartridge to the base unit.

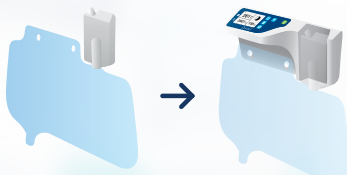


### 4. Connect drainage bag connector to catheter.



Note that Sippi will not register all urine if the bladder drains quickly. This initial volume in the bladder is normally clinically irrelevant.

### 5. To set the first hour – press and hold the sync button for at least 3 seconds.

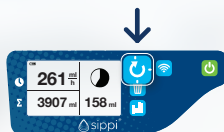

Syncing only needs to be done when starting a new patient.

## Sippi™ Quick Guide

# Continuous management of Sippi™

**For information on starting Sippi™ after connecting a new patient – see reverse.**

Sippi keeps track of hours and measuring days. Normally the only handling necessary is reading the hourly diuresis and resetting from the previous measuring day.

### Reading hourly diuresis

Read on Sippi's display.

Hourly diuresis,  
previous hour

Time indicator  
for current hour



Diuresis, current  
measuring day

Diuresis,  
current hour

### To start a new measuring day

Press and hold the rubbish bin button for at least 3 seconds, at any time during the first hour AFTER beginning a new measuring day.



The new measuring day is counted from the top of the first hour after reset. Sippi remembers this time the next measuring day.

### Remember!

- Sippi measures more accurately when hung straight.
- Sippi automatically divides up hourly urinary output into full hours.
- We do not recommend “milking” the tube.
- For other questions, read the user guide.

### History – current measuring day

Press the history button to scroll through and get an overview of the patient's diuresis so far in the current measuring day.

